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**EN WHAT IT IS**  
This hypertonic saline solution is a sterile preparation composed of salts and purified water. The vial is sterile and ready for use: easy to use and keep on hand. The disposable product is provided in single-dose single-patient vials. Use in a single administration in order to not alter the sterility of the product.

**INDICATIONS**  
The hypertonic saline solution is indicated for nasal irrigation. It cleanses the nasal cavities and decongests the upper

airways in the event of colds, rhinitis and sinusitis, helping drain secretions and foster proper conditions for carefree breathing.

**INSTRUCTIONS FOR NOSE CLEANING**

Twist the tab to open the sterile vial (fig. 1); lean your head to the left or, for children, lay the child down on one side supporting the head with your hand (fig.2), gently rest the opening of the vial against the upper nostril, press on the vial lightly and irrigate the nostril abundantly.

Wait a moment for the solution to act. Clear any secretions with a tissue. For children, lift the child and clear any secretions with a nasal aspirator or tissue (fig. 3). Turn the head and repeat the operation with the other nostril.

**COMPOSITION**  
2.5% sodium chloride, 1% sodium bicarbonate, 0.05% monosodium phosphate, purified water as required

**WARNINGS**  
Use as instructed by a doctor/paediatrician. For nasal use, do not ingest. Do not

inject. Store in a cool and dry place, do not expose to sources of heat and light. Do not use after the expiration date. The product is only sterile if the package is intact and undamaged. For children less than two months of age, or patients with significant respiratory illness, or in the event of frequent nose bleeds, or neurological or musculoskeletal diseases or maxillofacial trauma consult your physician before use. Store the instructions for as long as the product is used. Report any serious accident occurring in connection with the device to your competent authority and to the manu-

facturer. Keep out of reach of children. Do not use if the packaging is damaged.

Date of manufacture: first 4 digits of LOT (ex. 23039999; 23=2023; 03=March)

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PIKDARE S.p.A., via Saldarini Catelli 10, 22070 Casnate con Bernate (CO)-Italy